

## The Remodeling Face®

**Wider-looking eyes, more pronounced cheekbones, less marked wrinkles and a more clearly defined facial contour: some of the results achieved by the Remodeling Face®**

The Remodeling Face® uses state-of-the-art bio-electrotherapy for tailored beauty treatment. In order to optimize their effects, different Biologique Recherche treatments can be associated to the use of Remodeling Face®. Only a professional expertise, a thorough understanding of facial muscles, a precise methodology and a sense of volume and shape will allow to make the most of the Remodeling Face® functionality.

**An advanced bio-electronic treatment for tailored skin care.**

**The combination of four currents for immediate and lasting results**



The Remodeling Face® uses three types of electric current and one radiofrequency wave (high frequency)

- **Galvanic Current**, electric unidirectional continuous current, which improves bioavailability. The galvanic current cleans and purifies the epidermis by stimulating the saponification of fatty acids contained in the sebum (descaling). It also promotes the bioavailability of ionizable agents (ionization). Thanks to its vasomotor function, it changes the diameter of small blood vessels and thereby stimulates microcirculation.  
As the circulation in the blood and lymphatic systems increase, it allows better skin oxygenation, giving it tone and radiance.  
The skin looks smoother as superficial dead cells have been eliminated.  
Finally fine lines and wrinkles are reduced thanks to the penetration of hydrosolubles substances.
- **Medium Frequency Current modulated in Low Frequency Current**. This electric polarized unidirectional current, with myotonic and remodeling effects, is produced by an alternating current generator. This current, due to its rapid repetition and pulsations, does not cause heating or burning, but stimulates muscle tone in the face. The Medium Frequency current works the muscles harmoniously. The benefits of this additional stimulation are cumulative with each treatment. A real fitness session for the face !
- **Athermic Pulsed High Frequency Current**. This radio wave in the 3MHz to 30MHz range, dynamizes and balances the epidermal functions. The High Frequency Current represents the “more profound” and energizing phase of the treatment. It supplements the actions of the Galvanic and Medium Frequency currents, to achieve optimal regeneration.
- **Electroporation**: A high-frequency impulse current to make the upper layer of the epidermis permeable and help the active ingredients migrate into the cells by temporarily creating multiple wells into their surface.

## **Remodeling Face® Treatments, a subtle interplay between currents, serums and epidermis**

The Remodeling Face® combines the power of an electric current with the power of Biologique Recherche products, a combination that can reinvigorate the skin by stimulating epidermal cells.

These four currents can be used either separately or together, offering a wide range of personalized treatments for each “Skin Instant”. A subtle interplay is created between currents, serums and the epidermis. Precise adjustment of the parameters enables the remodeling process to be activated in both the skin and the muscles. Remodeling Face® and Biologique Recherche’s exclusive skin care products are complementary and improve one another’s performance.

Beauty treatments using Remodeling Face® can be divided into two stages.

### **1. A balancing stage to prepare the skin to absorb skincare serums**

The first step in every Biologique Recherche treatment is to bring the epidermis to a balanced state. This is achieved by successively using Cleansing Milk, Lotion P50 and Masks. The mask application can be combined with electric stimulation using the Remodeling Face®. This is the highlight of the Remodeling Face® treatment preparation stage.

The masks are chosen according to the skin’s moisture level, the tone of the tissue to be treated, the thinness of the skin and any particular problems such as sebum secretion, allergic sensitivities, etc...

### **2. A treatment stage, electro activation of Serums on the epidermal interface**

This regulation stage prepares the skin to receive the Quintessential Serums, and some of the Targeted Serums and/or Finishing Serums. Polypeptides and proteins in the Quintessential Serums are amphoteric substances that are either acids or bases. Under the influence of the electric field generated by the Remodeling Face®, these charged molecules migrate either at the positive or negative pole of the electrodes. The molecules solubilised in the serums are too large to allow these substances to penetrate, however, the electric field and the change in the pH of the local epidermis promotes the bioavailability of the peptides and polypeptides present.

## **The Remodeling Face® Results**

The Remodeling Face® helps remodel face, neck and décolleté with real action on texture and volume.

It helps reduce circles around eyes and tones the eyelids, making eyes look wider.

It improves the quality of the epidermis and muscle tone. The lifting effect is astonishing, immediate and lasting.

From the very first session, it visibly revitalizes the epidermis.

It also prepares skin tissue for medical-surgical aesthetic procedures and enhances the quality of post-operative results.

Finally it improves the penetration and the availability of chosen water-soluble active ingredients by temporarily reducing the resistance of the stratum corneum and creating multiple temporary pores in cell surfaces.

## **Contra indications**

The Remodeling Face® must not be used in the following circumstances : pacemaker, pregnancy, febrile state, fever, confirmed cancer. Locally it is not recommended in case of dental infection, mouth infection, skin infection, skin condition, sores, implants near cheekbones or eyes, Botox and fillers.